


Go Talk (Pocket, 9, 4, 20)

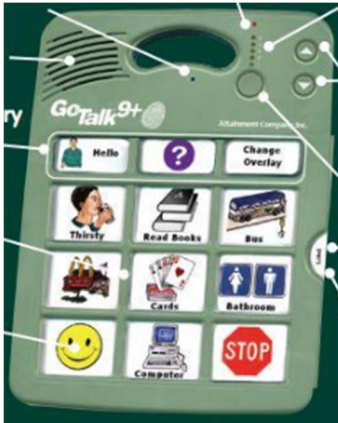

- 1) Press and release the RECORD button. The level indicator LED (green) and the recorder indicator LED (red) will light. While they are lit, press and release the MESSAGE key that you want to record. As you press the MESSAGE key, the record LED blinks to indicate recording.
- 2) When you finish the message, press any key to stop recording. Continue to record each message or record the entire level. When you've recorded all your messages (or the entire level or levels), press and release the RECORD button.

QuickTalker (7, 12, 23)



- 1) Press and hold the RECORD button on the back of the device until the red light is on, then release.
- 2) Press and hold the desired message location, begin speaking after the beep, then release. Repeat for other locations.
- 3) Press and release the RECORD button. Lights turn off. *Device will automatically exit record mode after 20 seconds of no activity.

Big Mack/Little Mack



BIG MACK

- 1) Press and hold RECORD until light appears.
- 2) Press and hold switch top while speaking, then release.
- 3) Press RECORD to finish.

LITTLE MACK

- 1) Press and hold both the record button and the switch top simultaneously.
- 2) Record your message then release both buttons.

Step-by-Step



- 1) Press the record button until red light appears
- 2) Press and hold the switch top
- 3) Record message, then release switch top
- 4) Repeat steps 2-3 until all messages are recorded
- 5) Press then release the record button

Big CHOICE with Levels



- 1) Press the hold RECORD until light appears
- 2) Select desired Level
- 3) Press and hold switch top while speaking, then release
- 4) Repeat step 2 for additional messages
- 5) Press RECORD to finish