

ProLoQuo2Go




1. From the Home page, tap MORE to move to next page.

2. Tap the EDIT button found on the bottom of the screen.



3. Select Create Folder. Select New Folder.

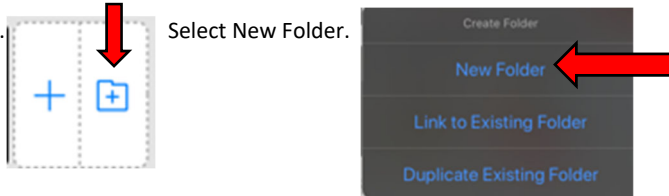


4. Choose a template.



5. Create folder name.

a. Add Picture to folder. Select symbol from bottom of the page or tap "Add Picture/edit" on folder symbol.

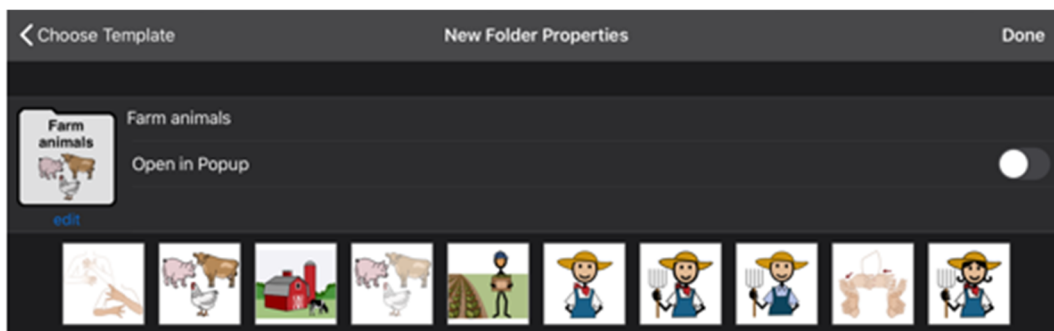

b. Select from the drop down menu. Create symbol or picture. Tap DONE


6. Tap DONE on top right of screen to return to Home page (2).

7. Open the folder created and tap the edit button (shown above).


8. Tap button to create on page.




- 9. Create button.
 - a. Add label.

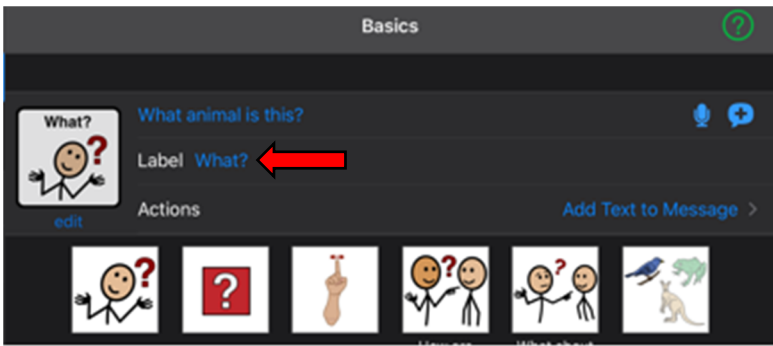

- b. Edit symbol on button.

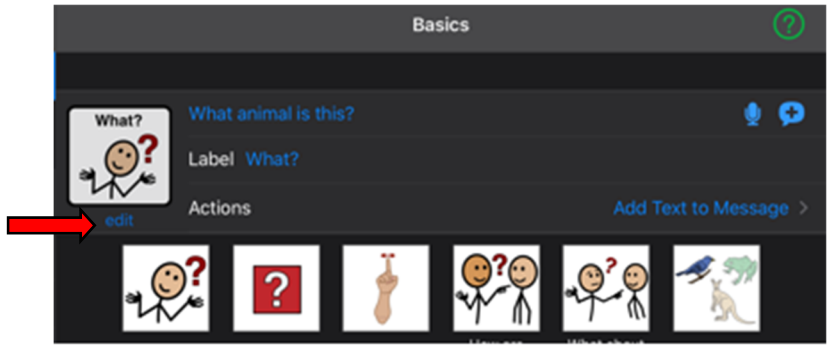

- c. Add speech to text by typing in word/words on line.

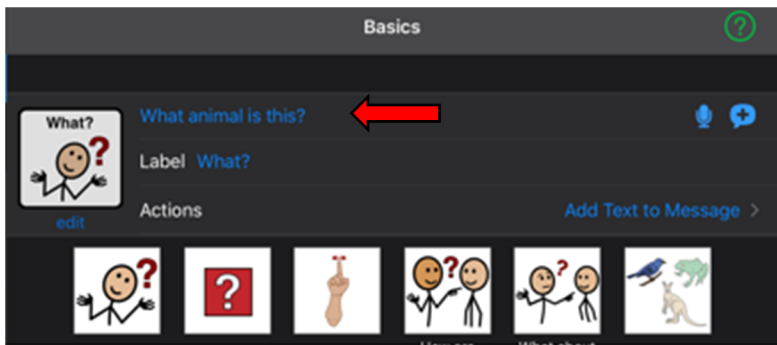

- 10. Repeat to create other buttons on page.
- 11. Tap DONE on top right of screen to return to Home page (2).