

TouchChat


1. TouchChat will open to the Vocabulary page. Tap ECSE PD.

| Lock | Name / Description | Version | Last Accessed |
|---|--|---------|--|
|  | 4 Basic 4-Basic offers just that, a Basic vocabulary option with 4 buttons per page. | v2.23.0 | |
|  | ECSE PD Modified January 24, 2020 at 11:34 AM  | v1.0 |  |

2. Tap ALL ABOUT ME.



3. Tap MENU.

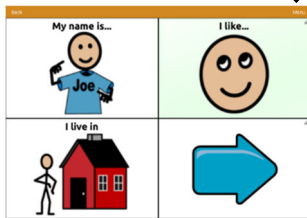

4. Tap EDIT PAGE.

5. Tap MY NAME IS. Tap EDIT THIS BUTTON.

- a) Edit Button Message.



- b) Edit image by SELECT IMAGE or IMPORT IMAGE.



- c) Tap SAVE on top right of box.

- d) Tap DONE on top left of page.

6. Tap button to hear edited button.
7. Repeat steps 2-6 to edit I LIVE IN button.
8. Tap I LIKE button.
9. Tap MENU button
10. Tap EDIT PAGE.
11. Tap blank button and repeat steps 2-6 to edit button.
12. Tap Done